

FROM ALLESEE ORTHODONTIC APPLIANCES

Roth

Gnathological Positioners

The objective of the Roth Gnathological Positioner is to optimize centric occlusion after debonding. The appliance is designed to be placed over the patient's teeth, hinging the mandible on the centric relation arc into the lower portion of the appliance. This allows the teeth to seat into the sockets without moving the mandible forward off the proper centric relation arc.

The key steps during fabrication:

- To move the occlusion closer to centric relation than it was at the time of debonding.
- To aid in providing better anterior guidance and posterior disclusion upon mandibular movements.
- To set up the teeth ideally without changing the relationship of the occlusal plane to the axis-orbital plane.
- To make sure that with proper construction the condyles are not subluxated from the fossae during positioner wear.

1-800-262-5221

Fax 1-414-886-6879

www.ormco.com/aoaapps.html

sds AOA
SYBRON DENTAL SPECIALTIES

P.O. Box 725
Sturtevant, WI 53177

AOA is a subsidiary of Ormco Corporation



Completed set up and positioner checked for centric relation. By using the Panadent articulator accuracy is easily transferred from the office to the laboratory.



Our exclusive Oralastic® silicone positioner material insures a perfect replication of freeway space and provides durability as well as elasticity.



Centric correlator used to transfer upper and lower set up relationship for the fabrication of the positioner.



AOA's other popular positioner materials are also available with this construction method.

What to send to AOA:

- Quality upper and lower models in stone or plaster mounted for use in adjustable articulator - preferably Panadent®
- Two piece power bite for verification of the mounting.
- Gnatho quick split magnetic mounting system may be utilized.
- A completed prescription form.

What AOA will return:

- Original mounted models and your wax bite.
- Mounted set up models with positioner.
- Carrying case and patient booklet "Your Positioner."
- Lifetime guarantee of the positioner against failure not related to patient abuse. Just return the set up and three piece bite.