

FROM ALLESEE ORTHODONTIC APPLIANCES

Active Vertical Correctors

Active Vertical Correctors utilize acrylic posterior bite blocks and incorporate repelling magnetic forces to accelerate the intrusive force needed to close anterior openbites.

Fabrication requires quality stone or plaster models with a construction bite reflecting 5mm of opening between the 1st molars. This amount of freeway space is needed to accommodate the magnets, casings and the acrylic splints.




Reprints of clinical cases are available upon request to our Customer Service number:



P.O. Box 725
Sturtevant, WI 53177

AOA is a subsidiary of Ormco Corporation

Rev. B

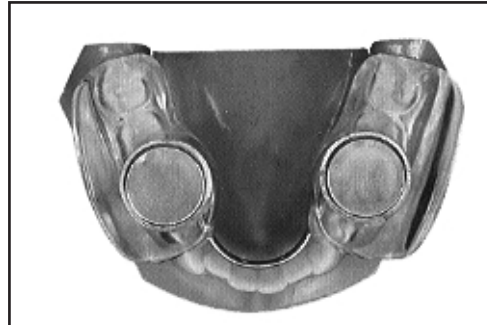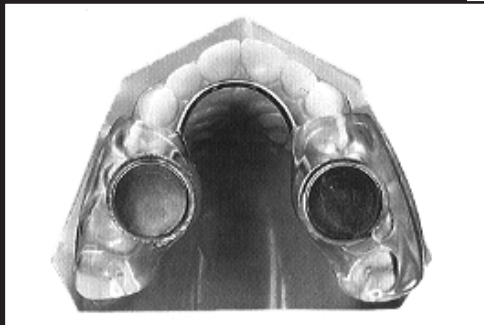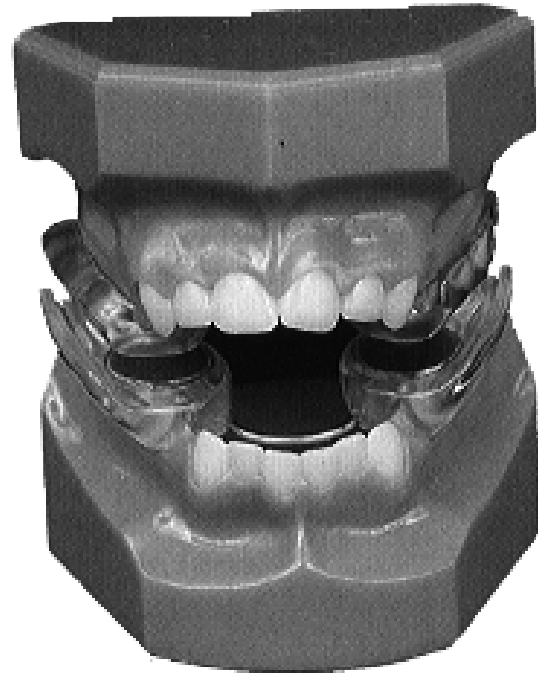

The magnets are encased in stainless steel and embedded into the acrylic splints. Angled buccal flanges are added to stabilize the appliance during lateral jaw movement.

Adjuncts such as upper expansion screw and claspings of choice can be added. A soft chin cup and strap are available for nighttime wear.

AVC 5mm Bite Guide

The AOA specially designed 5mm bite guide facilitates more precise bite opening. Available upon request.

1-800-262-5221

Fax 1-414-886-6879

www.ormco.com/aoaapps.html